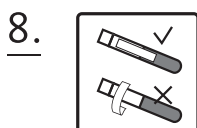
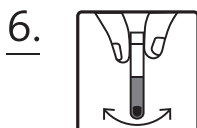
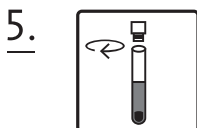
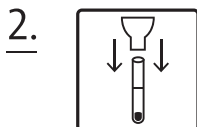
# SalivaGene Collector

## DEUTSCH

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### SalivaGene Collector - Instructions

**IVD** For In Vitro Diagnostic Use\*

\*) Compliance with EU Directive 98/79/EC on in vitro medical devices (Not for in vitro diagnostic use in countries where the EU Directive 98/79/EC on in vitro medical devices is not recognized.)

#### INTENDED USE:

Collection of human saliva samples for DNA stabilization and extraction (using the SalivaGene DNA Extraction Kits). The DNA in SalivaGene stabilized samples is stable for 1 year at room temperature within the expiry date.

The product is intended for use under supervision of professionals only, such as technicians, physicians and biologist trained in molecular biological techniques.

#### CONTENTS:

Funnel, tube with lid, each tube contains approx. 150 mg of SalivaGene reagent.

#### LIMITATIONS OF THE PROCEDURE:

Do not eat, drink, smoke or chew gum at least 30 min prior to saliva collection. Collecting 2 ml of saliva may require several minutes. Putting some grains of sugar on the tongue helps to produce saliva and does not interfere with the stabilization.

**1. Unscrew lid from tube and put it aside for further use.**

**2. Insert funnel into tube tightly.**

**3. Rub cheeks against teeth intensely for 30 sec.**

**4. Collect saliva to indicated fill level, avoid making and measuring air bubbles.**

**5. Remove and discard the funnel. Screw lid tightly onto the tube again.**

**6. Shake tube for 15 sec to dissolve white reagent.**

**7. Store the tube upright for 2-20 min with occasional shaking until Saliva Gene reagent is dissolved. Some cloudy material may occur during this process. This does not interfere with stabilization.**

**8. For barcode sample tracking stick the small barcode tape vertically onto the tube.**

**STORAGE:** Store at room temperature 15 – 30°C (59 - 86°F).

#### SHIPMENT:

Ship collected saliva according to the instructions of SalivaGene Collector provider. Ship according to locally applicable guidelines and regulations such as UN 3373 (Biological Substance, Category B).

#### SAFETY PRECAUTIONS:

Keep out of the reach of children. Small cap: choking hazard. Do not remove SalivaGene reagent from the tube. Do not use the SalivaGene Collector if content of the tube is spilled. Do not ingest the contents of the tube. Wash with water if SalivaGene reagent or dissolved SalivaGene reagent mix comes in contact with skin or eyes.

#### WARNING:

**H319:** Causes serious eye irritation.

**H411:** Toxic to aquatic life with long lasting effects.

**P280:** Wear protective gloves/protective clothing/eye protection/face protection.

**P305+P351+P338:** IF IN EYES: Rinse cautiously

with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

#### LABEL LEGEND:

Consult operating instructions

Store at room temperature 15 – 30°C.

Do not reuse

Expiry date

Lot number

1035211100

CE Marking

In vitro diagnostic medical device

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Invitek Molecular GmbH