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## ENGLISH

### SalivaGene Swab Comfort – Instructions

CE  For In Vitro Diagnostic Use\*

*\*) Compliance with EU Directive 98/79/EC on in vitro medical devices (Not for in vitro diagnostic use in countries where the EU Directive 98/79/EC on in vitro medical devices is not recognized.)*

#### INTENDED USE:

Collection of human oral samples for DNA stabilization and extraction (using the SalivaGene DNA Extraction Kits).

The DNA of swab samples collected using SalivaGene Swab Comfort is stable for at least 6 months at room temperature in the stabilization buffer. Long-term storage of the sample (> 6 months) should be done at - 20° C.

The product is intended for use under supervision of professionals only, such as technicians, physicians and biologist trained in molecular biological techniques.

#### CONTENTS:

Swab, Stabilizer tube, label for donor description. The Stabilizer tube contains approx. 650 µl of liquid DNA Stabilizer reagent.

#### LIMITATIONS OF THE PROCEDURE:

Do not eat, drink, smoke or chew gum at least 30 min prior to sample collection. Ensure the swab tip is not exposed to any surface prior to collection. Put the Stabilizer tube upright to prevent the liquid inside the tube from spilling. Be sure to move the swab over the entire cheek and to moisten it with saliva.

#### SAFETY PRECAUTIONS:

Keep out of the reach of children. Small cap: choking hazard. Caution should be used when inserting swab into the mouth. Do not use the SalivaGene Swab Comfort if content of the tube is spilled. Do not drink the DNA Stabilizer reagent. Wash with water if DNA Stabilizer reagent comes in contact with skin or eyes.

#### WARNING:

**H319:** Causes serious eye irritation.

**H412:** Harmful to aquatic life with long lasting effects.

**P280:** Wear protective gloves/protective clothing/eye protection/face protection.

**P305+P351+P338:**

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

See MSDS at [www.invitek-molecular.com](http://www.invitek-molecular.com)

## INSTRUCTIONS:

1. Open swab package and remove swab without touching the tip.

2. Insert swab into the mouth and rub swab tip firmly against the inner cheeks for about 30 seconds on each side.

3. Unscrew lid from Stabilizer tube and put it aside for further use. Insert swab into the Stabilizer tube.

4. Break swab at breaking point. Discard the broken handle part.

5. Screw lid tightly onto the Stabilizer tube again.

6. Gently shake tube for 15 sec to mix buccal cells and DNA Stabilizer reagent.

7. Label Stabilizer tube with donor name and collection date.

## LABEL LEGEND:

-  Consult operating instructions
-  Store at room temperature (15 – 30 °C)
-  Do not reuse
-  Expiry date
-  REF 1035231100
-  LOT Lot number
-  CE CE Marking
-  IVD In vitro diagnostic medical device

Invitek Molecular GmbH  
Robert-Rössle-Str. 10  
13125 Berlin, Germany

+49 30 9489 2901

[info@invitek-molecular.com](mailto:info@invitek-molecular.com)

[www.invitek-molecular.com](http://www.invitek-molecular.com)

**Attention:** Swab sample storage in stabilization buffer may cause slightly yellowish color. This has no effect on the extraction process and downstream applications.

## SHIPMENT:

Ship collected swab samples according to the instructions of product provider. Ship according to locally applicable guidelines and regulations such as UN 3373 (Biological Substance, Category B).

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